


HOW TO TETHER THE SPIRIT™ APS™

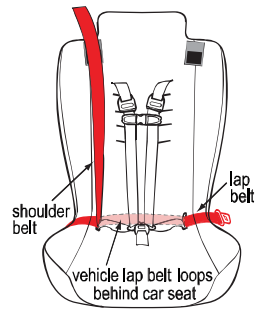


If you install with
Vehicle **LAP/SHOULDER BELT**

Spirit APS Tether Strap **MAY** be required

Vehicle **LAP BELT** only


Spirit APS Tether Strap **IS** required

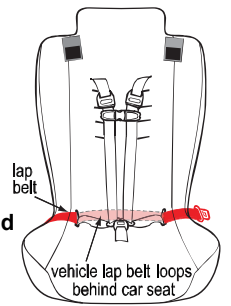

1. Determine if vehicle shoulder belt is suitable as top tether


1. Determine if vehicle has tether anchorage

If the vehicle shoulder belt lies **WITHIN** the "Shoulder Belt Path" labels on the car seat...

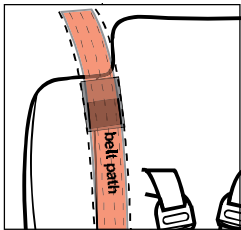


You **MAY** use the shoulder belt as the top tether.

If the vehicle shoulder belt **DOES NOT** lie within the "Shoulder Belt Path" labels...

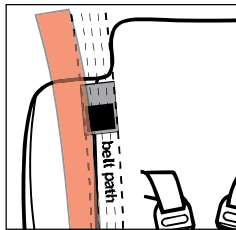

Use the **DUAL HOOK** tether strap as the top tether.

If the vehicle shoulder belt **DOES NOT** lie within the "Shoulder Belt Path" labels **AND** your vehicle **DOES NOT** have tether anchorages...

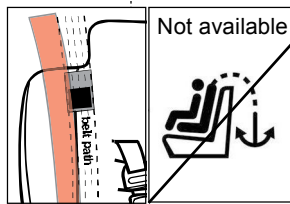

Order the Spirit heavy duty bolt-in tether hardware kit. Have the kit installed by a qualified technician. Then use the **DUAL HOOK** tether strap as the top tether.

Yes



No




Order the Spirit heavy duty bolt-in tether hardware kit. Have the kit installed by a qualified technician. Then use the **DUAL HOOK** tether strap as the top tether.

Skip to step 3


2. Determine tether hook anchorage(s)

2. Determine tether hook anchorage(s)


Use **ONE** hook for individuals **LESS** than 80 lbs. (36 kg)

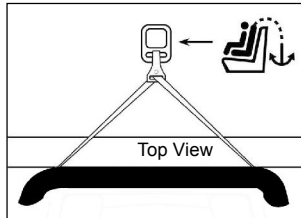

Use **TWO** hooks for individuals **80 lbs. (36 kg) or MORE.**

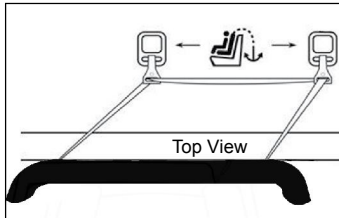


Attach **TWO** hooks on the tether bracket (see 2400P-02 bolt-in instruction)

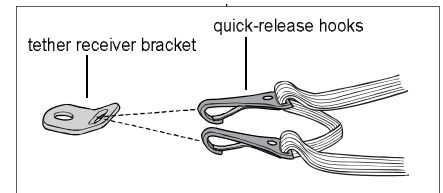

3. Secure the car seat

Make sure the vehicle lap or lap/shoulder belt is tight and in locked (ALR) mode (consult your vehicle manual for seat belt locking information). If a tether strap is used, tighten the adjusters to secure the car seat. Test the car seat to make sure it cannot be moved from side to side more than 1 inch at the belt path.

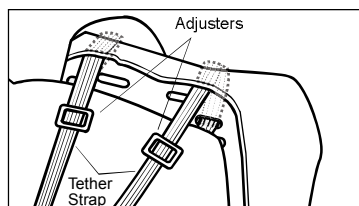

Columbia
MEDICAL
Solutions for Daily Living