


## 8825 Assembly Instructions

(caster installation onto PVC extension legs)

### Step 1

Remove the rubber feet from base of chair or extension legs.

Fig. 1

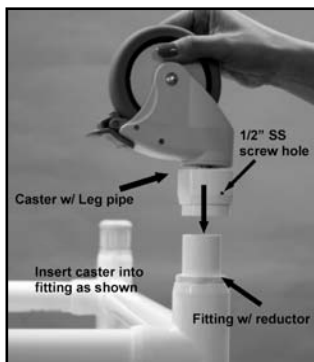

### Step 2

Insert caster leg assembly into the socket legs. See Fig. 1.

*NOTE: Caster assembly must be fully inserted into the leg support to prevent breakage.*

### Step 3

Install (1) 1/2" stainless steel screw in each leg on pre-drill holes. See Fig. 1.

Fig. 2



### Step 4

Place your Columbia bath chair on top of the extension legs.

### Step 5

Firmly press the bath chair legs into the extension legs as far as they will go. There are four predrilled placement holes for the screws. With the bath chair legs set, fasten the screws to lock the bath chair legs into the extension legs. See Fig. 2.

Columbia Medical

13577 Larwin Circle • Santa Fe Springs, CA 90670

(800) 454-6612 • Fax: (310) 305-1718 • Email: info@columbiamedical.com

Website: www.columbiamedical.com

© 2008 Columbia Medical Mfg., LLC

*COLUMBIA MEDICAL MFG. LLC warrants this product to be free of defects in material and workmanship. Our obligation under this warranty is limited to repair or replacement of any part or entire unit at our option for a period of one year from date of delivery to the original purchaser. The warranty does not include cost of inconvenience, property damage, misuse, abuse, accident or similar incidents. The warranty will not apply if product has been tampered with or repaired by unauthorized individuals.*

C0000-XX-XX

Rev. B (Updated: 10/08)

**Columbia**<sup>™</sup>  
MEDICAL

*Solutions for Daily Living*

## PVC Extension Legs Instructions

*For use with Deluxe<sup>™</sup> and Supreme<sup>™</sup> PVC Bath Chairs*

### Model Numbers:

8850 (raises bath chair height by 11 inches)

8860 (raises bath chair height by 15 inches)



8850 shown above



# ATTENTION

Your Columbia PVC Bath Chair Extension Legs have been built to the highest standards of quality to ensure years of superior service. Please take the time to read this informative instruction guide and pay particular attention to the safety information provided.

Here are some additional precautions you can take to prevent the risk of injury:

1. Read and understand ALL instructions.
2. Determine, with a physician or therapist, the suitability of this product for the intended user.
3. NEVER leave a user unattended while in the bath chair and/or in the bath.
4. Check regularly that all tubing, bolts, buckles, and other parts are tight at ALL times.
5. When occupied, make sure the user is buckled in at all times.
6. Chair legs should be NEVER be in complete vertical positions.
7. SAVE THESE INSTRUCTIONS.

If you require additional assistance, please call Columbia Customer Service at **(800) 454-6612**, or e-mail [info@columbiamedical.com](mailto:info@columbiamedical.com)

**Thank you for choosing Columbia.**

## Before You Begin

The Extension Legs are designed to be used with your Columbia™ Contour Deluxe™ and Supreme™ Bath Chairs. The Extension Legs raise the height of your bath chair in the tub. Advantages of a higher bath chair include easier transfers and a reduced risk of back strain. As the bath chair will be above bath water level, you will need a hand-held shower hose in your tub for proper bathing.

## 8850 & 8860 Assembly Instructions (installation onto Columbia™ PVC Bath Chair)

### Step 1

After unpacking the extension legs, identify the side of the frame that features 2-inch (5cm) PVC tubes protruding from the four corners.

### Step 2

Remove the rubber feet from the Columbia bath chair and slide them over the ends of the tubes mentioned in Step 1. This will serve as the base that sits on the bottom of the extension legs. Set rubber feet on the floor.

**NOTE for 8860 only:** Use the color-coded dots to determine proper assembly. It is important to note that the four tubes can only be installed one way. A push button will hold in the adjacent tube.

### Step 3

Remove plastic bag containing the four screws.


### Step 4

Place your Columbia bath chair on top of the extension legs.

### Step 5

Firmly press the bath chair legs into the extension legs as far as they will go. There are four predrilled placement holes for the screws. With the bath chair legs set, fasten the screws to lock the bath chair legs into the extension legs.

*Insert bath chair legs here*

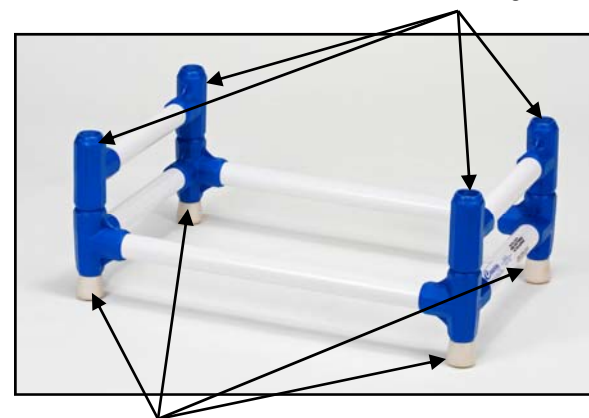

*Rubber feet from chair transferred to extension leg tubes*