



Contour™ PVC Bath Chair Instructions

For Reclining & Tilt-in-Space Models

Deluxe™ Models #8500, 8600, 8700

Supreme™ Models #8510, 8610, 8710



Thank you for Choosing Columbia

Your Columbia Contour Deluxe/Supreme Bath Chair has been built to the highest standards of quality to ensure years of superior service. Please take the time to read this informative instruction guide and pay particular attention to the safety instructions provided. Thank you again for choosing Columbia!

Important Safety Instructions

When using your bath chair, basic safety precautions should always be followed to reduce the risk of injury to persons.

1. Read and understand all instructions.
2. Determine, with a physician or therapist, the suitability of this product for the intended user.
3. Never leave a user in your Contour Deluxe/Supreme Bath Chair unattended in the bath, even for a moment. Your Contour Deluxe/Supreme Bath Chair provides support but does not replace adult supervision while bathing.
4. Again, when this support is used in or around water, uninterrupted supervision is necessary to reduce the risk of injury or drowning.
5. Check regularly that all bolts, buckles and other parts are tight at all times.

Save These Instructions

Assembly of Your Contour Reclining Bath Chair

Different size Contour Deluxe/Supreme Bath Chairs assemble differently.

Small Contour Deluxe/Supreme Bath Chair Assembly

The small Contour Deluxe/Supreme Bath Chair is shipped with the seat not attached to the frame. Using two bolts and nuts provided, attach the seat section to the frame. One bolt is used on each side. Reach inside the seat cover, by opening the flap on the top of the mesh. From the inside of the seat, stick the bolts through the holes in the plastic tubing, and then through the holes in the plastic frame. The cap nuts should be screwed to the bolt ends on the outside of the frame. If they are on the inside of the seat the user may feel them under the mesh cover. Next, insert the tube at the rear of the seat back into the larger tube on the frame. Press the metal release button to insert the tube.

Medium Contour Deluxe/Supreme Bath Chair Assembly

On the medium Contour Deluxe/Supreme Bath Chair, the seat is already bolted to the frame. All you need to do is insert the tube at the rear of the seat into the larger tube on the frame. Press the metal release button to insert the tube.

Large Contour Deluxe/Supreme Bath Chair Assembly

On the large Deluxe model bath chair is shipped with the seat not bolted to the frame. Bolt the seat section to the frame, with one bolt on each side first. Then, insert the tubes under seat front into the larger tube on frame. Press the metal release button to insert the tube.

For Supreme Models

Follow above instructions for the size of the chair, and then insert the tube under seat front into the larger tube on the frame. Press the metal release button to insert the tube.

On the large Supreme Model Bath Chair, the seat is already bolted to the frame. All you need to do is insert the tubes at the rear of the seat into the larger tubes on the frame. Press the metal release button to insert the tube.

Setting the Angle of Recline of the Bath Chairs – Deluxe or Supreme Models

Simply push the release button on the upright post. Slide the inner post up or down, until the button locks in the desired position. (The large chair has two such posts.)

Setting the Seat Angle of the Contour Bath Chairs – Only on Supreme Models

Push the release button on the post under the seat. Slide the inner post up or down, until the button locks in the desired position. (The large chair has two posts).

Other Uses

Between baths, use your Contour Deluxe/Supreme Bath Chair outdoors on the lawn, in an outdoor wading pool, or indoor as a go-anywhere bath chair. The Supreme Models can lay flat for convenient storage between usages.

Cleaning Your Contour Deluxe/Supreme Bath Chair

Soapy water or mild cleansers may be used on all parts of your bath support. Harsh abrasive cleaners (Ajax, Comet, etc.) may dull the finish. You do not need to worry about rust, as the metal parts are stainless steel.

Columbia Medical
13577 Larwin Circle
Santa Fe Springs, CA 90670
☎ (562) 282-0244
Fax: (310) 305-1718
Email: info@columbiamedical.com
Website: www.columbiamedical.com
© 2007 Columbia Medical Mfg., LLC

COLUMBIA MEDICAL MFG. LLC warrants this product to be free of defects in material and workmanship. Our obligation under this warranty is limited to repair or replacement of any part or entire unit at our option for a period of one year from date of delivery to the original purchaser.

The warranty does not include cost of inconvenience, property damage, misuse, abuse, accident or similar incidents. The warranty will not apply if product has been tampered with or repaired by unauthorized individuals. This warranty gives you specific rights which vary from state to state.